

rad-sure™ gamma blood irradiation indicators

indicators for positive visual verification of irradiation

description

Rad-Sure™ blood irradiation indicators provide positive, visual verification of irradiation at the minimum specified dose. Manufactured from Gafchromic™ film, the world's highest resolution dosimeter, Rad-Sure™ has been the standard for blood irradiation indicators for over 25 years. Before a blood product and its attached indicator are irradiated, the indicator reads "NOT IRRADIATED." After the blood product and its attached indicator are irradiated, the word "NOT" is obscured and the indicator reads "IRRADIATED." Rad-Sure™ gamma irradiation indicators should be used with irradiators that have cesium-137 or cobalt-60 radiation sources or other sources producing radiation of equal or greater energy.





benefits

- not subjective—the product has been properly irradiated when the film turns black and "NOT" is completely obscured
- color-blind friendly—no need to match colors, just read the indicator to know the blood product has been irradiated
- made with accurate and reliable film used in radiation oncology centers around the world
- uses a powerful adhesive to help the indicator stick and stay on blood bags
- easy to use: just peel, stick, irradiate, and read
- U.S. FDA listed medical device
- ISBT 128 bar-coded lot numbers
- minimum dose of 15 Gy or 25 Gy available

product list

product	product code
rad-sure™ 15 Gy blood irradiation indicators	831377
rad-sure™ 25 Gy blood irradiation indicators	831379

examples

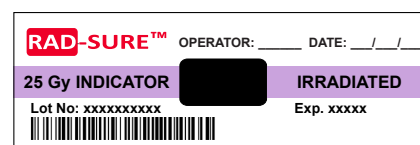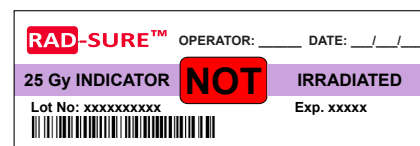

Rad-Sure™ gamma irradiation indicators before and after irradiation.