



Mammographic Film/Screen Contact Test Tool

GAMMEX 157A

Film/Screen contact can have a significant influence on image quality.

Today's mammography film/screen systems have higher resolution than diagnostic radiography x-ray systems and require test tools with finer detail. Regular testing with the Gammex 157A detects problems and artifacts early. Use of the tool and the evaluation of the resulting images is simple. Areas of poor screen contact appear darker than areas of good contact. Any dark areas greater than 1 cm should be investigated and corrective action taken. Gammex recommends testing cassettes very six months or when new or repaired cassettes are put into service.

SPECIFICATIONS

Screen size.....	24x30 cm (9.4x11.8 in)
Mesh	#40 Mesh -C copper
Size.....	25.8x31.5 cm (10.2x12.4 in)
Weight.....	0.4 kg (0.9 lbs)



WWW.GAMMEX.COM

PEO B.V.

info@gotopeo.com
www.gotopeo.com

The Netherlands

Havenweg 16, 6603 AS Wijchen
+31 (0)24 648 86 88

Belgium

Watermolenstraat 2, B-2910 Essen
+32 (0)3 309 32 09

CoC 34107894

VAT NL807859151B01

IBAN NL29 RABO 0356 1960 46

BIC RABONL2U



PEO B.V.

info@gotopeo.com
www.gotopeo.com

The Netherlands

Havenweg 16, 6603 AS Wijchen
+31 (0)24 648 86 88

Belgium

Watermolenstraat 2, B-2910 Essen
+32 (0)3 309 32 09

CoC 34107894

VAT NL807859151B01

IBAN NL29 RABO 0356 1960 46

BIC RABONL2U

